

KNOW THE DEMON YOU'RE DEALING WITH IN UNDER 20 MINUTES!



*It adds up to a...
healthy calf*

Know what you're fighting,
with Ark Quatro Detection Kit
and win the war on scour.

ARK QUATRO DETECTION KIT

Ready to use kit for detection of
Cryptosporidium, **Rotavirus**,
Coronavirus & **Escherichia coli K99**

- **Specific and sensitive**
- **Easy to perform**
- **Simple and accurate**
- **Easy to interpret**
- **Result in 15 mins**

Eradicating scour from any farm is notoriously difficult owing to the number of ways it is caused, but if you know what's causing scour on your farm, you can deal with it. And that's where Ark Quatro Detection Kit comes in. It's a simple '5 Step' on-farm test kit that allows you to test for the cause of scour yourself in under 20 minutes. Ark Quatro Detection Kit will identify if it is one of the 4 main causes of scour you have on your farm - Rotavirus, Ecoli, Cryptosporidium or Corona virus. So keep one handy during the calving season, and know the devil you're dealing with.

How to use:

STEP 01

Scoop a scour sample, no more than ½ a level spoonful and place a sample in each of the 4 tubes.

DO NOT EXCEED ½ A LEVEL SPOONFUL PER TUBE.

Result will be compromised, if exceeded.



STEP 02


Stir each sample into buffer provided, using the spoon.
Allow to stand for 2 mins after stirring.

DO NOT SHAKE.



STEP 03

Place an indicator strip in each of the 4 tubes.
Run the 4 tests simultaneously.

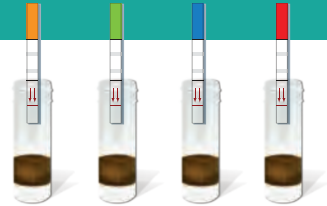

STEP 04

Wait 15 minutes for results.



STEP 05

Compare the results on each strip against the performance test chart (as illustrated) to determine results.

Positive: 2 red lines



Negative: 1 red line



Invalid: No lines



NB: Ensure that the sample used is taken from an untreated animal.



For more technical information: Ark Animal Care, Newbridge Industrial Estate, Newbridge, Co. Kildare
T: (353) 45 433 203 | F: (353) 45 433 934 | E. info@arkanimalcare.ie | www.arkanimalcare.ie